Product information – Cosmea organic tampons

Cosmea organic tampons are made from 100% organically grown cotton. They contain no animal ingredients and are produced without animal testing (certified by The Vegan Society). The skin compatibility has been dermatologically assessed as “very good”. With a proportion of 99.7% renewable raw materials, they contribute to the conscious use of natural resources.

Cosmea Bio tampons suitable for the FemCare FC1 feminine hygiene dispenser

Menstrual products such as tampons and sanitary towels are just as much a part of the washroom today as water, soap and towels. The new 3-in-1 solution for feminine hygiene products is an all-rounder: it holds sanitary bags, up to 30 pads and 160 tampons in a single, compact dispenser.

Product features:

- Tampon made of 100 % organic cotton
- Certified according to STANDARD 100 by OEKO-TEX®
- Absorbent core for efficient fluid absorption
- Free of synthetic fiber wrapping
- Return ribbon made of 100 % organic cotton
- Hygienically individually packaged
- 99.7% renewable raw materials
- Dermatologically rated “very good”
- Vegan – without animal ingredients and animal testing

For comfort, safety & a good feeling

Surface with grooved structure

Surface with grooved structure
Comfort and well-being thanks to even expansion inside the body


Reliable protection

All-round safety thanks to efficient fluid absorption and distribution in the tampon core

Retrieval straps made of 100 % organic cotton

200× twisted for particularly easy removal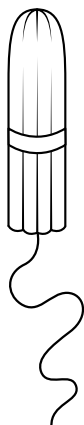

Tampon made of 100% organic cotton

Certified according to STANDARD 100 by OEKO-TEX® and GOTS by Ecocert Greenlife; Guaranteed pesticide-free cultivation

Hygienically individually packaged

Particularly thin organic protective film made from 90 % renewable raw materials with practical tear strip

Absorbent core without synthetic fiber coating

100 % nature out of love for the environment and Respect for women's nature

Instructions for use

Wash your hands thoroughly before inserting the tampon. Carefully remove the packaging and insert the tampon. The tampon should be changed regularly, at the latest every 4 to 8 hours

Important information

- The tampon should be changed after 4 to 8 hours at the latest.
- Only one tampon may be used at a time.
- Tampons should only be worn during menstruation
- They should always be kept clean and dry

Toxic Shock Syndrome (TSS)

TSS is a rare but potentially serious disease caused by the bacterium *Staphylococcus aureus*. Possible symptoms include a sudden onset of high fever, dizziness, vomiting, diarrhea, skin rash or muscle pain. If TSS is suspected, seek medical help immediately and remove the tampon immediately.

Sizes and absorbencies

- Mini – for light days
- Normal – for normal days
- Super – for heavier days
- Super Plus – for very heavy days

Disposal

Tampons must be disposed of with the residual waste. Do not dispose of in the toilet.

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EN Instruction for use

You should always use **the minimum absorbency** required to match your flow.

1. Wash your hands and tear the protective foil open as indicated by the arrow. Remove **bottom** part of the protective foil.
2. Unwrap the tampon string and pull it quickly and firmly. Make sure both ends of the string are connected.
3. Push your index finger into the hollow at the end of the tampon and remove **the top part** of the foil.
4. Insert the tampon into the vagina; this will be easiest if the body is relaxed (sitting or standing with one leg up on a chair). Push the tampon into the vagina as far back as possible, at a slight upward angle. The tampon is inserted correctly when you cannot feel it anymore.
5. The tampon can be removed by slowly pulling the string downwards and forwards. If you feel resistance, the tampon might not be soaked enough. Tampons can be changed as often as needed but should not be left longer than 8 hours. Wash your hands after removal of the tampon. Do not flush it down the toilet - it may cause trouble in the sewage system. Please dispose used tampons in an appropriate way, along with your household rubbish.

If you cannot find the string for any reason, wait for the tampon to be completely saturated. Then squat and press heavily. The tampon will move downwards and you will be able to grab it easily. Please consult your gynaecologist if you fail to remove the tampon. As COSMEA® tampons can hardly be felt, always make sure to remove the old tampon before inserting a new one. **Never use more than one tampon at a time. At the end of your period, make sure that you have removed the last used tampon.**

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EN Toxic shock syndrome

Toxic shock syndrome (TSS) is a rare, but very serious illness that can happen to everybody. It is caused by toxins produced by bacteria that is commonly found. Approximately 50 % of all TSS cases have been reported by menstruating women and girls. TSS has been associated with the use of tampons, but can also occur during or after your period without using tampons. **As TSS can be fatal, it has to be treated early.**

The following symptoms can occur suddenly, at any time, during or shortly after your period – and do not have to occur at the same time: Sudden high fever (39 °C or higher), skin rash, diarrhea, sore throat, dizziness, fainting or vomiting > remove the tampon immediately and **visit your doctor**. Tell the doctor about TSS and your period. If you develop TSS, ask your doctor for advice on further use of tampons.

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