Productinformatie – Cosmea biologische tampons

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EN Instruction for use

You should always use **the minimum absorbency** required to match your flow.

1. Wash your hands and tear the protective foil open as indicated by the arrow. Remove **bottom** part of the protective foil.
2. Unwrap the tampon string and pull it quickly and firmly. Make sure both ends of the string are connected.
3. Push your index finger into the hollow at the end of the tampon and remove **the top part** of the foil.
4. Insert the tampon into the vagina; this will be easiest if the body is relaxed (sitting or standing with one leg up on a chair). Push the tampon into the vagina as far back as possible, at a slight upward angle. The tampon is inserted correctly when you cannot feel it anymore.
5. The tampon can be removed by slowly pulling the string downwards and forwards. If you feel resistance, the tampon might not be soaked enough. Tampons can be changed as often as needed but should not be left longer than 8 hours. Wash your hands after removal of the tampon. Do not flush it down the toilet - it may cause trouble in the sewage system. Please dispose used tampons in an appropriate way, along with your household rubbish.

If you cannot find the string for any reason, wait for the tampon to be completely saturated. Then squat and press heavily. The tampon will move downwards and you will be able to grab it easily. Please consult your gynaecologist if you fail to remove the tampon. As COSMEA® tampons can hardly be felt, always make sure to remove the old tampon before inserting a new one. **Never use more than one tampon at a time. At the end of your period, make sure that you have removed the last used tampon.**

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EN Toxic shock syndrome

Toxic shock syndrome (TSS) is a rare, but very serious illness that can happen to everybody. It is caused by toxins produced by bacteria that is commonly found. Approximately 50 % of all TSS cases have been reported by menstruating women and girls. TSS has been associated with the use of tampons, but can also occur during or after your period without using tampons. **As TSS can be fatal, it has to be treated early.**

The following symptoms can occur suddenly, at any time, during or shortly after your period – and do not have to occur at the same time: Sudden high fever (39 °C or higher), skin rash, diarrhea, sore throat, dizziness, fainting or vomiting > remove the tampon immediately and **visit your doctor**. Tell the doctor about TSS and your period. If you develop TSS, ask your doctor for advice on further use of tampons.

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